



L I F E C O M E S F I R S T

ARCHEON Medical
archeon-medical.com

2 Chemin des Aiguillettes
25000 BESANCON, FRANCE

Tel : +33 (0)3 81 66 23 80
contact@archeon-medical.com

A0000092-Rev01-30NOV2021

EOLifeX


CPR: Better training to save lives



1.

Connecting / disconnecting the FlowSense X sensor to EOLife X

- o Insert the part with a notch in the connector ridge.
- o Press the sensor firmly to make contact with the pins of the connector.
- o A distinctive “click” sound should be heard when the tab on the connector holds the FlowSense X sensor in position.
- o In order not to damage the connector, **pull** the tab with the arrow marking to unclip the FlowSense X sensor from the EOLife X housing.

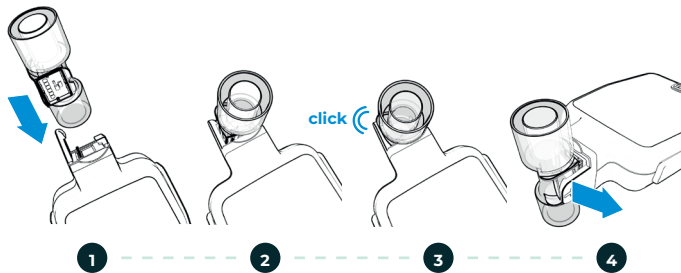

2.

Start of EOLife X

Press the physical ON / OFF button once to switch on EOLife X



3.

Selection of operating mode



4.

Training mode

Choose the training mode



5.1

Blind interface

Record indicator
Click stop to stop the recording



5.2

Monitored ventilation

Advanced interface

- Real time feedback
- Vi = insufflated volume
 - Vt = tidal volume
 - Freq. = ventilation frequency

Bargraph (Vi in real time)
Alarm signal display area



Basic interface

Ventilation instructions
Bargraph (Vi in real time)
Alarm signal display area

6.

Statistical report

Ventilation data recording and export
Statistical report at the end of the ventilation

Regulatory certification

Hereby, ARCHEON declares that the radio equipment type EOLife X is in compliance with Directive 2014/53/EU.
The full text of EU declaration of conformity is available at the following link:

Contains FCC ID: SQGBL652
Contains IC: 3147A-BL652



For more information:

- Consult the EOLife X instruction for use via Flash Code
- Watch the training video on our Youtube account Archeon Medical

